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
Shielded IV Catheters

Before Activation

After Activation

2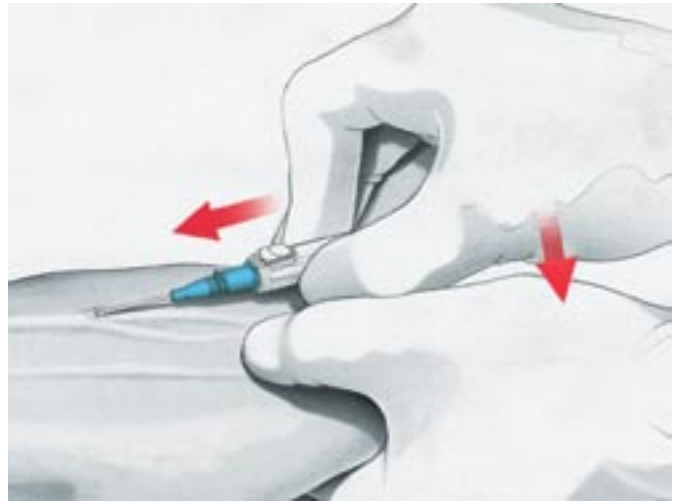

POINTS TO PRACTICE

1 PREPARATION

- Prepare all equipment prior to venipuncture
- Make sure all items are accessible throughout the procedure
- Prepare site according to policy and procedure

2 VENIPUNCTURE

- Approach vein slowly at a low angle
- Observe flashback. In 22 and 24 GA flashback can initially be seen along the catheter

3 ADVANCEMENT

- Upon flashback visualization, lower catheter almost parallel to the skin
- Advance *entire unit* slightly before threading catheter
- Thread catheter into vein while maintaining skin traction

4 NEEDLE REMOVAL


Before Pressing the Button

- Release tourniquet
- Apply digital pressure beyond the catheter tip
- Gently stabilize catheter hub
- Press the white button


5 SECUREMENT

- Secure catheter and apply sterile dressing according to policy and procedure

3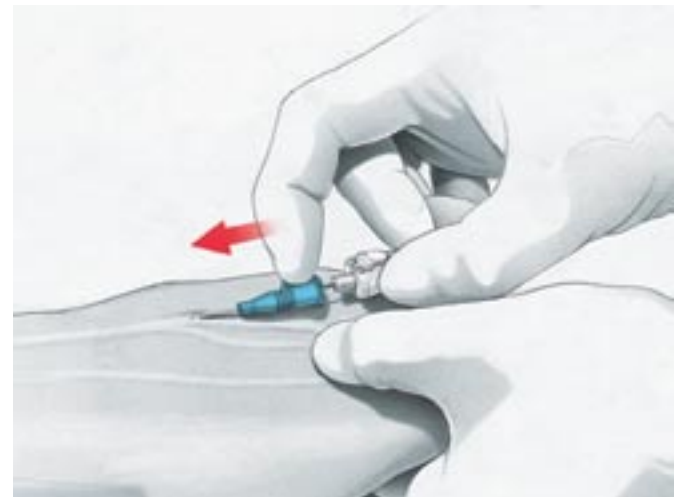

4

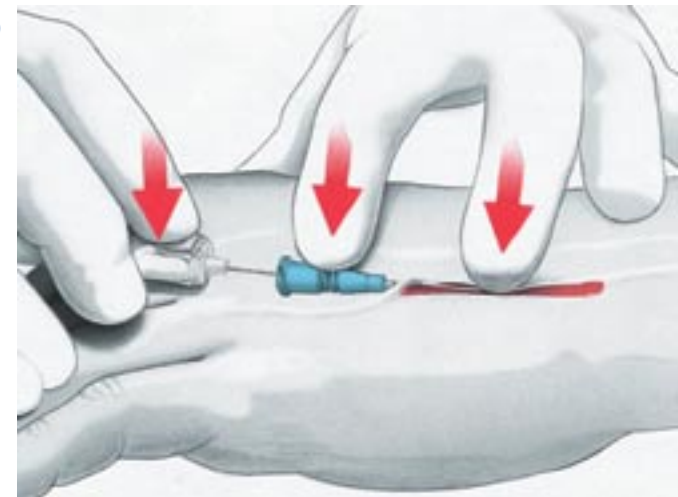CAUTION REMINDERS

- **Do Not** withdraw needle from catheter hub before pressing the white button.
- **Needle should be retracted prior to disposal** in a puncture-resistant, leak-proof sharps container.
- **Never Reinsert Needle** into the catheter as this could shear the catheter.
- **Do Not Use Scissors** at or near the insertion site.

BD Medical Systems

Sandy, Utah 84070

1-888-237-2762, www.bd.com/infusion

D13779A (11-00)

BD, Angiocath and Autoguard are trademarks of Becton, Dickinson and Company.

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