


BD Integra™ Syringe

w/ Retracting BD PrecisionGlide™ Needle

QUICK REFERENCE CARD

1

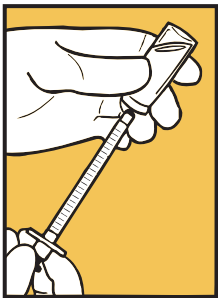

ASPIRATION

Draw up medication in accordance with established protocol.

2



INJECTION

Administer medication in accordance with established protocols, making sure that all medication has been dispensed.

BD Integra™ Syringe

w/ Retracting BD PrecisionGlide™ Needle

QUICK REFERENCE CARD


3



ACTIVATION

Push plunger rod to activate the safety mechanism. Depress until you sense two "clicks."

4

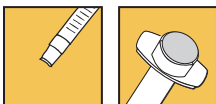

RETRACTION

The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete. There is no exposure to the used needle. Immediately discard into an approved sharps container.

Things to Remember:

- BD Integra Needle and Syringe are not a standard luer connection.
- After pressurizing the medication vial, do not continue to press the plunger rod past the point of resistance, as this may cause premature activation.