

GRIPPER[®] Micro Blunt Cannula, Non-Coring Safety Needle



Non-sharp blunt cannula remains in port



Small, low-profile infusion site for patient comfort



Safe and comfortable

A revolutionary design that fits your patient's lifestyle

GRIPPER® Micro Safety Needle

The unique safety and comfort features of the popular GRIPPER PLUS® needle have now been incorporated into the new GRIPPER® Micro blunt cannula, non-coring safety needle, along with several new additions that make it ideal for small patients or for those requiring long-term or repeated infusions.

The new GRIPPER® Micro safety needle is a two-piece system that includes an inserter and an infusion site. After the needle is inserted into an implanted port, the inserter (sharp) is removed, leaving behind a small, low profile infusion site with a blunt cannula. The needle's unique safety arm provides an audible CLICK when it is lifted for extra assurance that the needle is locked into a safe position. And the infusion site's small size and foam pad provide increased patient comfort for long-term therapy.



GRIPPER® Micro safety needle offers:

- > Small, low profile infusion site for ease in dressing
- > Nonabsorbent foam pad for extra patient comfort
- > Non-sharp blunt cannula for safe removal and disposal following infusion
- > Trocar, non-coring needle that locks into place for sharps disposal

Easy and Safe Insertion



Insert needle into port.



Press tab in and lift safety arm until it clicks, then dispose of inserter and safe needle into a sharps container.



All that's left is a small, low profile infusion site.

Actual size...
smaller
than a dime



ORDERING INFORMATION

Gauge	Length	Reorder Numbers Non Y-site	Reorder Numbers NAC
19	3/4 inch	21-3260-24	21-3280-24
19	1 inch	21-3261-24	21-3281-24
20	3/4 inch	21-3256-24	21-3276-24
20	1 inch	21-3257-24	21-3277-24
22	3/4 inch	21-3251-24	21-3271-24
22	1 inch	21-3252-24	21-3272-24

THE DETAILS GIVEN IN THIS LEAFLET ARE CORRECT AT THE TIME OF GOING TO PRESS. THE COMPANY RESERVES THE RIGHT TO IMPROVE THE EQUIPMENT SHOWN

Smiths Medical MD, Inc.

St. Paul, MN 55112, USA
Phone: 1-800-426-2448
www.smiths-medical.com

Smiths Medical, part of the global technology business Smiths Group

Customer and Clinical Services 1-800-426-2448
www.smiths-medical.com

GRIPPER, PORT-A-CATH, and the Smiths Medical design mark are trademarks of the Smiths Medical family of companies. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. ©2008 Smiths Medical family of companies. All rights reserved. 9/08 192501

smiths medical