

Insertion



Access a port

Prepare the site according to facility protocol. Holding on to the inserter as shown, insert the needle into the port.



Finger on tip



Press tab in



Lift safety arm

Remove the needle

- From the back of the inserter, place fingers on each side of the inserter's base to stabilize it.
 - With the other hand, place a finger on the tip of the inserter's safety arm.
 - **Press the tab in** and lift the safety arm straight back until the needle **CLICKS** into the locked position.
- ✓ **The sharp is now removed.**



Dispose in sharps container

- Dispose of the inserter in a sharps container.
- Apply a semi-permeable dressing over the infusion site, ensuring that a minimum 4cm area surrounding the site is covered.

Removal



Remove infusion site

Place fingers on each side of the infusion site. Stabilize the port with the other hand. Lift the infusion site straight up and discard per facility protocol.



4/08 192502

Quick
Reference
for access and de-access

Blunt Cannula, Non-Coring Safety Needle

GRIPPER[®] Micro

Customer & Clinical Services

1-800-426-2448

www.smiths-medical.com

smiths medical

Smiths Medical MD, Inc.

St. Paul, MN 55112, USA

Phone 1-800-426-2448

www.smiths-medical.com

Smiths Medical, part of the global technology business Smiths Group

Deltec[™]

GRIPPER, Deltec, and the Smiths Medical design mark are trademarks of the Smiths Medical family of companies. ©2008 Smiths Medical family of companies. All rights reserved. 4/08 192502