

BD Bard-Parker™ Protected Disposable Scalpel

featuring locking retractable shield

Protection
Without
Compromise



Indispensable to
human health



Introducing

BD Bard-Parker™ Protected Disposable Scalpel

featuring locking retractable shield

Protect Yourself. Protect Your Staff. Designed to minimize the risk of surgical blade injuries, the BD Bard-Parker™ Protected Disposable Scalpel gives clinicians, throughout the hospital...

Protection Without Compromise

- **In Procedure:**
Permitting clinicians to focus first on the surgical site, rather than on the instrument.
- **In Handling:**
Providing instrument characteristics of weight, handling, and feel that clinicians prefer.
- **In Technique:**
Allowing clinicians to retract the protective shield, easily and safely with one hand.
- **In Reliability:**
Assuring quality and performance you have come to expect from the BD Bard-Parker™ brand.





- **Clear Protective Shield**
- **Locks when blade is covered** ▶ Protection before and after use
- **Locks when blade is uncovered** ▶ Security and control during use
- **One-handed activation** ▶ Ease of use
- **Transparent** ▶ Blade visible

- **Top Button**
- **Tactile sensation** ▶ Confirms lock
- **Audible click** ▶ Confirms lock

- **Lock Indicator Line**
- **Visual reference** ▶ Confirms lock

- **Surgical Green Plastic Handle**
- **Autoclave Compatible** ▶ Allows on-site sterilization

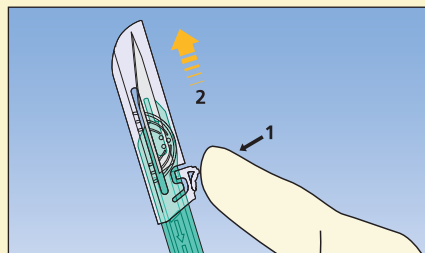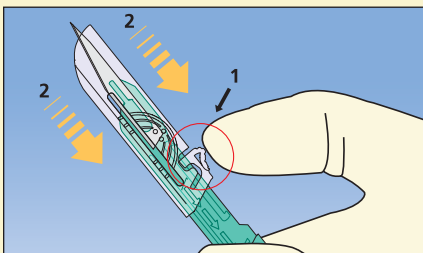

Sterile Protected Scalpels:

372610	# 10	
372611	# 11	
372612	# 12 *	
372613	# 12B *	10 Disposable Scalpels per box,
372615	# 15	10 boxes
372620	# 20 *	per case
372621	# 21 *	
372623	# 23 *	*Available 2003

Protection with Ease and Simplicity

For preparation, peel open package and dispense scalpel onto sterile field.

To Use

1. Depress top button on protective shield.
2. Retract shield to rear locked position to fully uncover blade.

To Cover

1. Depress top button on protective shield.
2. Advance shield to forward locked position at lock indicator line.

Dispose of protected scalpel in an approved sharps container after use.

With BD Bard-Parker™, clinicians have a total, integrated system of products, services, programs, and professional support that enhance and contribute to safer, surgical practices. Another reason why BD is **First in Safety**.

BD Bard-Parker™ Protected Disposable Scalpels may be ordered through your BD Distributor. For information call: 1.800.268.5430 or visit www.bd.com


Customer Service
1.800.268.5430



BD Medical Systems
2771 Bristol Circle
Oakville, ON L6H 6R5
www.bd.com

BD Bard-Parker™ Protected Disposable Scalpel

featuring locking retractable shield

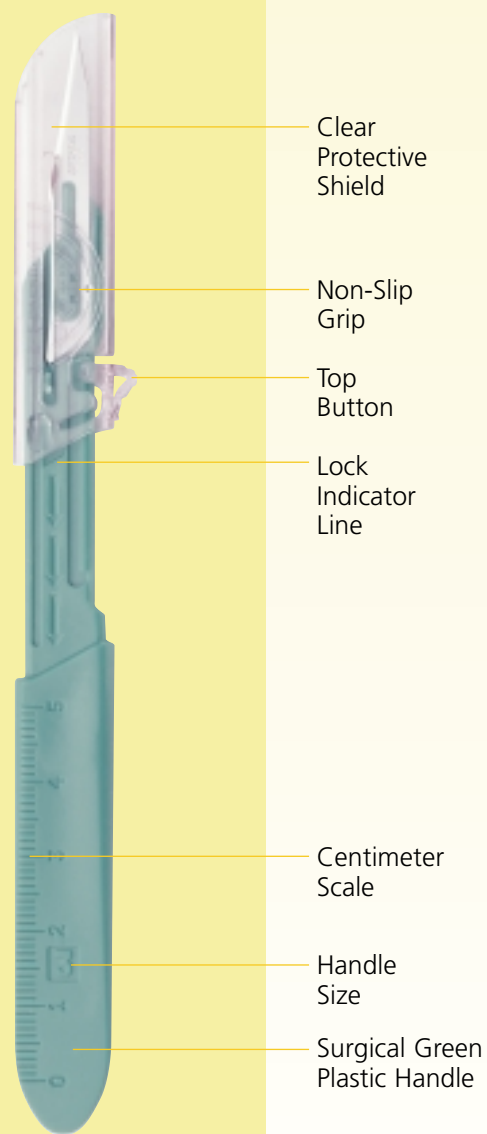

Clear
Protective
Shield

Non-Slip
Grip

Top
Button


Lock
Indicator
Line

Centimeter
Scale

Handle
Size


Surgical Green
Plastic Handle

protected

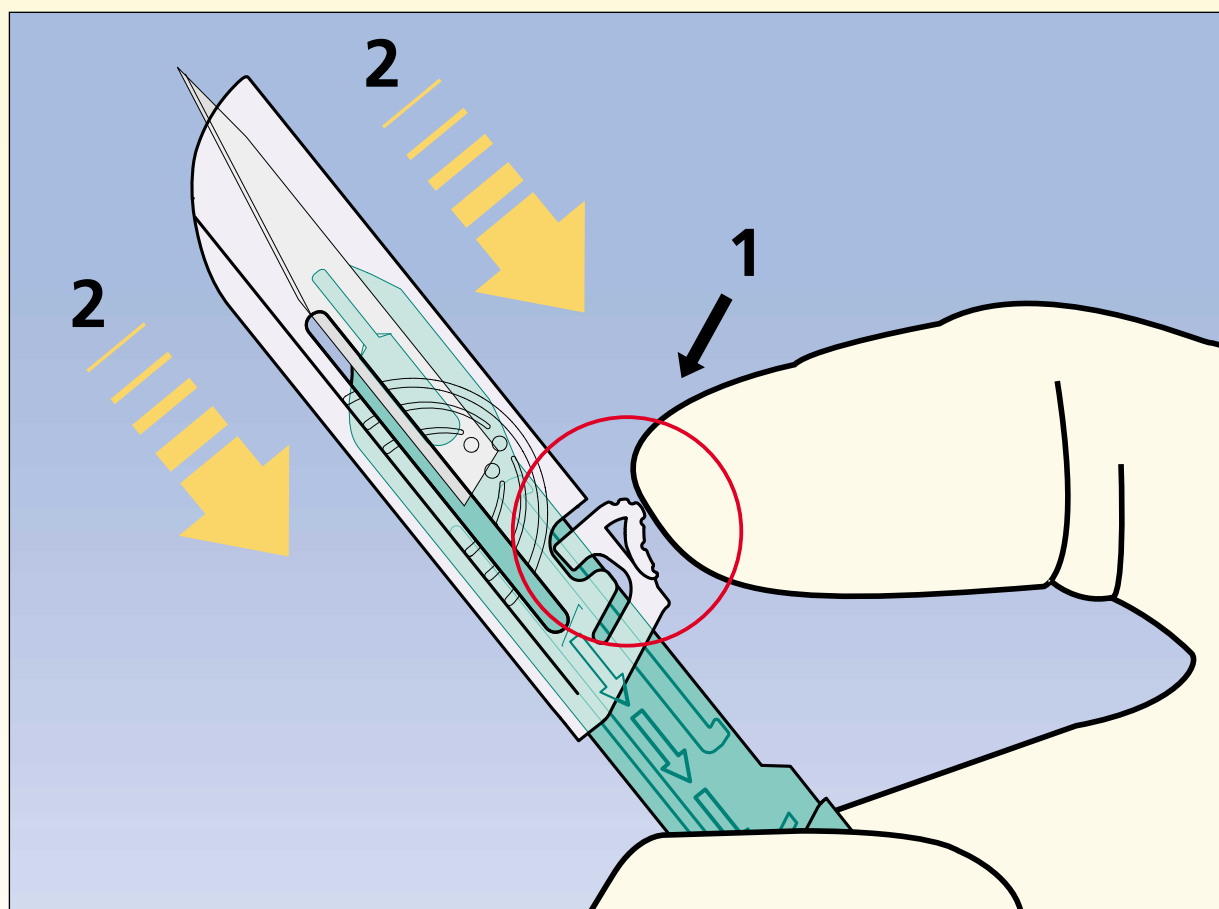


To Use

1. Depress top button on protective shield.
2. Retract shield to rear locked position to fully uncover blade.

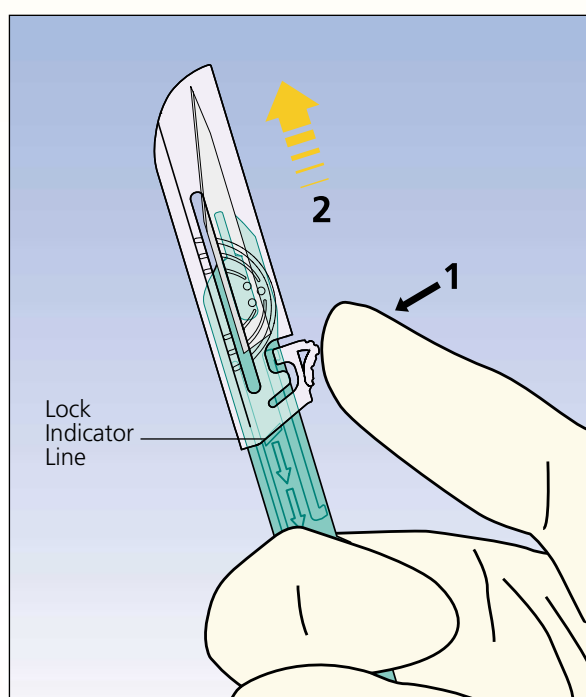

To Cover

1. Depress top button on protective shield.
2. Advance shield to forward locked position at lock indicator line.

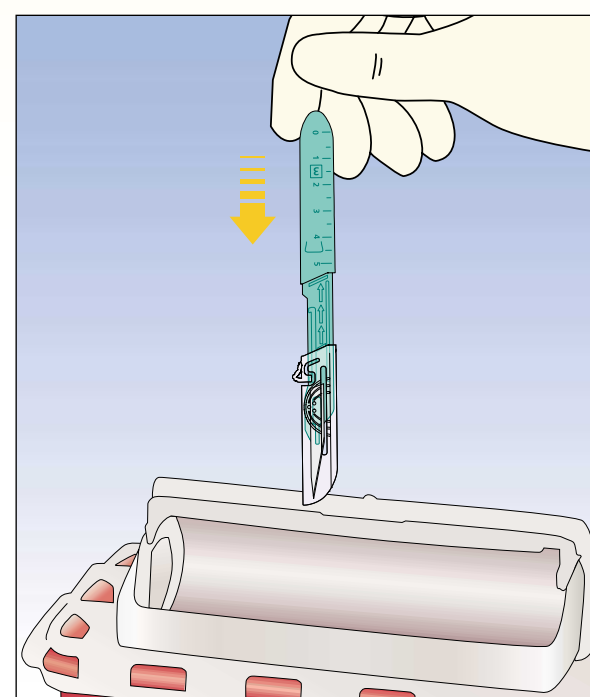

Disposal

Dispose protected scalpel in an approved sharps container.

BD Bard-Parker protected disposable scalpels are designed to minimize the risk of surgical blade injuries that occur before use, during passing, between steps of the procedure, after use and during disposal. The protective shield is easily activated with one finger by depressing the top button to slide it either to its "in use" or "protected" position. The scalpels are virtually the same size and weight as conventional scalpels.

Call today for your free Inservice Safety Survival Kit: 1.800.817.1702 or visit www.bd.com/scalpel through 2001.

Protection
without
Compromise



BD Bard-Parker
stainless steel blade

in use